CATTANI GROUP

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FAST&STERIL

Single-dose blister pack for cold sterilization with peracetic acid solution. The blister contains 2 components and makes 2,5 litres of solution.

Application: Surgical instruments and dental aspirator.

When to use: The prepared solution lasts 36 hours. Before this time it can be used over and over, on clean, dry instruments or it can be sucked up through the dental aspirator using the "Pulse Cleaner" bucket.

How to use: Pour the entire content of a blister pack in 2.5 litres of water at 35/40°C. Mix until dissolved and then let it rest for 10 minutes. For the next 36 hours (2 working days) the solution will sterilize in 15 minutes (extract and rinse the sterilized instruments in an aseptic way).

Range of action: Tests have shown that the solution has a bactericidal, fungicidal, tuberculocidal, virucidal (all viruses) and also sporicidal (sterilizing) action in 15 minutes.

Certificate: CE 0434 Medical Device Class 2a

Reference rules:

- UNI EN 14561
- UNI EN 14562
- UNI EN 13704
- UNI EN 14563
- UNI EN 14476

High-level disinfectant-detergent with orthophthaldehyde for the initial cleansing treatment after use of surgical instruments. 1 litre package.

Application: ALL METAL/PLASTIC SURGICAL INSTRUMENTS

When to use: Immediately after use.

How to use: It is a concentrate to be diluted for use at 1,5% or 3%. The solution is active for 14 days and after that it can be disposed of in the sewage system.

Range of action:

- Diluted at 1,5% it is proven disinfectant within 1 hour.
- Diluted at 3%:
 - bactericidal (proven effective within less than 5 minutes)
 - fungicide (proven effective within less than 10 minutes)
 - tuberculocide (proven effective within less than 5 minutes)
 - virucide (proven effective within less than 5 minutes)

Certificate: Class IIa - Medical Devices

Reference rules:

- UNI EN 13727
- UNI EN 13624
- UNI EN 14348
- UNI EN 14476
- CE 0434
- UNI EN 14561
- UNI EN 14562
- pr EN 14563

Concentrated sterilizer with orthophthaldehyde liquid for the preparation of sterilizing solutions for surgical instruments.

1000 ml bottle with measuring cup.

Application: CLEAN AND DRY SURGICAL INSTRUMENTS OF WHATEVER MATERIAL

When to use: For the treatment of surgical instruments which cannot be sterilized thermally.

How to use: Pour the solution into a basin and dip the instruments completely. After the sterilizing time (8 hours for a solution diluted at 1,5% and 4 hours for a solution diluted at 6%), extract the instruments aseptically and rinse with sterile water. It can be reused for 14 days after checking with special indicator and then disposed of in the sewage system.

Range of action: Bactericide, fungicide, tuberculocide, virucide (against all viruses). The product is also sporicidal (sterilizing action) within 8 hours if diluted at 1,5% and 4 hours if diluted at 6%.

Certificate: CE 0434 Class IIa - Medical Devices

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Certificati: CE 0434 Disp. Med. Classe IIa

QUICK STERIL INDICATOR

Accessory to verify the Quick Steril sterilizing efficacy - Bottle for 100 tests.

Application: FRESHLY PREPARED OR REUSED QUICK STERIL STERILIZING SOLUTIONS

When to use: To be used before the sterilizing treatment to ensure that the minimum effective concentration (MEC) of the Active Ingredient is present in the solution.
How to use: Aspirate 1 cm³ of indicator and 1 cm³ of the solution to be tested into the syringe. Wait 2 minutes before comparing the colour of the liquid inside the syringe with the colour of the table printed on the label.

Certificate: CE 0434 Class IIa - Medical Devices

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High-level disinfectant-detergent wet wipes with didecyl dimethyl ammonium chloride.

Package contains 100 wipes.

Application: FLAT AND SMOOTH SURFACES, HANDLES, ...

When to use: After every intervention.

How to use: Ready-to-use.

- Wipe clean with a gentle pressure the surface to be treated.
- Let it work for 5 minutes. Self-drying.

Range of action:

- Detergent
- Disinfectant:
 - bactericida (proven effective within less than 5 min.)
 - virucida (HBV - HCV under 10 minutes. HIV under 15 minutes)
 - fungicida (proven effective within less than 15 minutes) • tuberculocida

Characteristics:

- It leaves no residue.
- Cool, flower-scented perfume.
- It contains less than 10% alcohol and is particularly mild on the chair covering material.

Precautions:

- It is generally safe on hands, it can be used without wearing gloves.
- It has been tested on many materials. As a precaution, we recommend carrying out a preliminary test on a part which is not in sight.

Certificate: Class IIa - Medical Devices

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High-level surface disinfectant-detergent with didecyl ammonium chloride

Application: IRREGULAR, ROUGH OR DIFFICULT TO REACH SURFACES

When to use: After every intervention.

How to use: Ready-to-use.

- Spray onto the surface from 20/30 cm.
- Let it soak for 5 minutes.
- Wipe with a dry sterile cloth.

Range of action:

- Detergent
- Disinfectant:
 - bactericida (proven effective within less than 5 min.)
 - virucida (HBV - HCV under 10 minutes. HIV under 15 minutes)
 - fungicida (proven effective within less than 15 minutes) • tuberculocida

Characteristics:

- Cool, flower-scented perfume.
- It contains less than 10% alcohol and is particularly mild on the chair covering material.

Precautions:

- It is generally safe on hands, it can be used without wearing gloves.
- It has been tested on many materials. As a precaution, we recommend carrying out a preliminary test on a part which is not in sight.

Certificate: Class IIa - Medical Devices

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Ready-to-use spray cleaner for medical autoclave chambers comes with "All Clean" cloth.

500 ml package.

Application: STAINLESS STEEL AUTOCLAVE CHAMBERS

When to use: When needed.

How to use: Ready-to-use.

Spray onto the surface to be treated, wipe clean, with the included microfiber cloth, rinse with water and dry.

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Certificati: CE 0434 Disp. Med. Classe IIa

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- UNI EN 13624
- UNI EN 14348
- Pr EN 13623

Concentrate for disinfectant solutions with bronopol and biguanide to prevent biofilm growth and cross infections.

500 ml bottle.

Application: DENTAL UNIT WATER PIPING (DUW)

When to use: During long work stops (week-end, holidays) or the pause between one treatment and the other.

How to use: Dilute the product at 4% and fill up the DUW. Waiting time: 5÷10 min.

Range of action: Diluted at 4% the product is legionellcidic within less than 5 minutes, tuberculocide and bactericide within less than 10 minutes and fungicide within less than 60 minutes. The product is safe on all the materials of the DUW.

Certificate: CE 0434 Class IIa - Medical Devices

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Spray silicone lubricant with non-flammable ozone-friendly propellant

Application: SURFACES EXPOSED TO RUBBING, RUBBER "O"-RINGS, PLASTIC AND METAL MATERIALS

When to use: When needed.

How to use: Ready-to-use.

- Shake before use.
- Clean and wipe the parts to be lubricated, then spray.

Range of action: Lubrication.

Characteristics:

- It contains high-quality silicone oil.
- Not flammable.
- Dust-repellent.

Precautions:

- Protect against sunshine.
- Do not expose to temperature > +50 °C.
- Do not spray onto naked flame or red-hot material.
- Do not swallow nor spray into eyes.

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Silicone lubricant for the periodic lubrication of the hinge of pincers, scissors and other instruments of the dental surgery

Application: SURGICAL INSTRUMENTS with parts subjects to friction (scissors, pincers,...)

When to use: After washing or in case of need, once or more times a day.

How to use: Dribble a drop on the parts under friction. To be used on clean and dry instruments before sterilization.

Range of action: Lubrication

Characteristics:

- It contains high-quality silicone oil.
- Dust-repellent.

Precautions:

- No particular storage precaution is required.
- Avoid contact with eyes.

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Application of the chemical products for the the hygiene of the dental surgery

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Magnolia is part of the Cattani group which is a leader in the production of dental aspiration systems. Nobody better than the manufacturer can suggest suitable disinfectants for his products.

Via Natta, 6/A

43122 PARMA - ITALIA

Tel. +39 521 607.604

Fax +39 521 399.967

www.cattani.it/magnolia

e-mail: info.magnolia@cattani.it

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If you do not have the "Guide to dental surgery disinfection" in your own language please ask Cattani S.p.A.